




Thank you for purchasing Auto Shoe Cover Dispenser. Please read all the instructions carefully before using it.

Name of parts

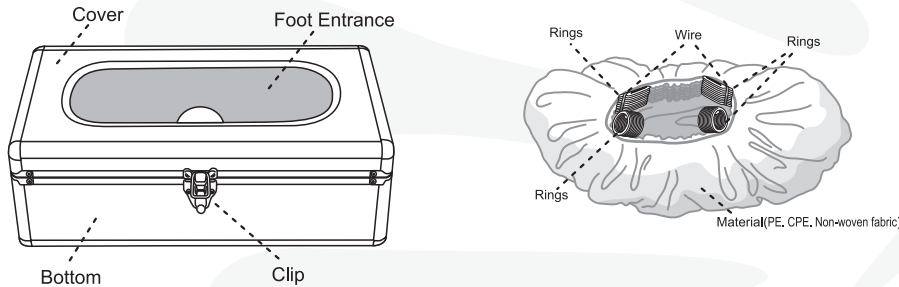


Instruction Menu

1. Release dispenser clip and open cover, you will find 2 “pushing bars” accompanied with blue releasing button.

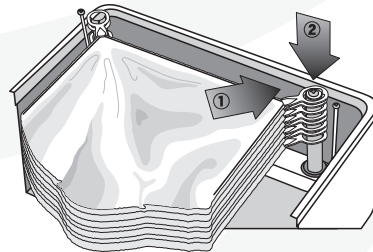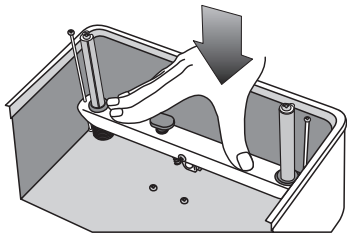


2. Before installing covers, please press “pushing bar” all the way down to the bottom, you will hear “click” then “pushing bar” will be held there.

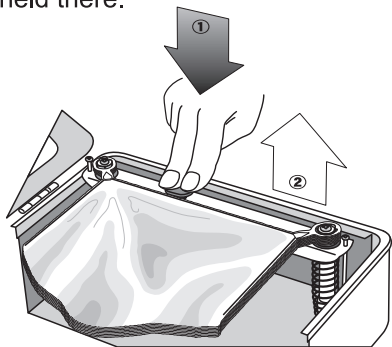

3. Every 10pcs shoe covers (the 5 plastic rings) are bounded by wire.....don't release the wires, just put whole bundle rings into the post 1, 2, 3, 4. The maximum capacity is 5 bundles (=50 pcs shoe covers). Then remove the wire easily with your fingers.

4. Press the “blue button” to release the “pushing bar”, it will then be pushed up and give support to shoe covers for releasing

Usage of auto shoe cover dispenser

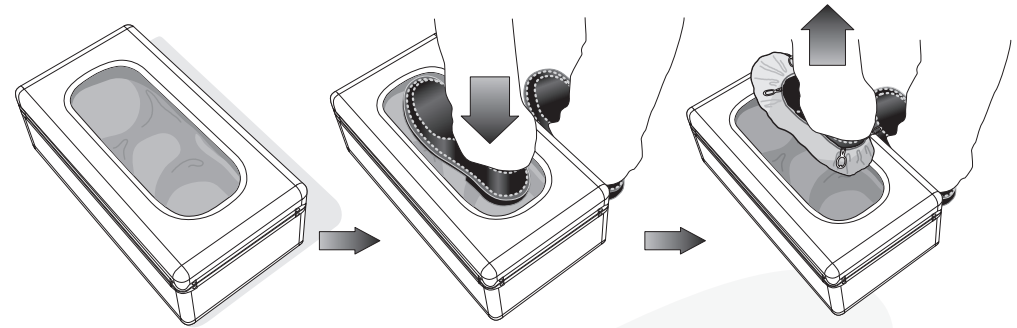

Use proper strength and quickly step on to centre of shoe cover dispenser, then shoe cover will be released immediately.

Most Seen Problems

Problem: One or more plastics clip does not come out after stepping down	
Possible reasons	Solutions
The strength of stepping is not even for all four corners. So, the plastic clip of one or more corner does not come out.	Step on the board evenly with strength.
The angle of the exit piece is too low.	Adjust the angle of the exit piece.
The screw of that pole is loosened.	Tighten the screw and adjust the position of pole.
Bungee cord for the shoe cover has broken.	Take that shoe cover out of the dispenser.
Binding strand has not been un-tied.	Get rid of binding strand.

Problem: More than one plastics clip comes out after stepping down	
Possible reasons	Solutions
The angle of the exit piece is too high.	Adjust the angle of the exit piece.
The screw of that pole is loosened.	Tighten the screw and adjust the position of pole.
Step on the board twice.	Step on the board only once every time.